SONY®

REF 4-533-378-03 (1)
2017-09

HD/D1-SDI Input Adaptor

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Before operating the unit, please read this manual thoroughly and retain it for future reference.

Symbols on the product

This symbol indicates the manufacturer, and appears next to the manufacturer's name and address.

This symbol indicates the EU Importer, and appears next to the EU Importer's name and address.

This symbol indicates the European Community representative, and appears next to the European Community representative's name and address.

This symbol indicates the serial number.

This symbol indicates the version of the accompanying document.

Caution

For cautions and further information on using this product, refer to the instructions for use for the monitor.

For customers in the U.S.A.

This equipment has been tested and found to comply with the limits for a Class A digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference when the equipment is operated in a commercial environment. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instruction manual, may cause harmful interference to radio communications. Operation of this equipment in a residential area is likely to cause harmful interference in which case the user will be required to correct the interference at his own expense. You are cautioned that any changes or modifications not expressly approved in this

manual could void your authority to operate this equipment.

All interface cables used to connect peripherals must be shielded in order to comply with the limits for a digital device pursuant to Subpart B of part 15 of FCC Rules.

For the customers in the U.S.A.

SONY LIMITED WARRANTY - Please visit <http://www.sony.com/psa/warranty> for important information and complete terms and conditions of Sony's limited warranty applicable to this product.

For the customers in Canada

SONY LIMITED WARRANTY - Please visit <http://www.sonybiz.ca/pro/lang/en/ca/article/resources-warranty> for important information and complete terms and conditions of Sony's limited warranty applicable to this product.

For the customers in Europe

Sony Professional Solutions Europe - Standard Warranty and Exceptions on Standard Warranty.

Please visit <http://www.pro.sony.eu/warranty> for important information and complete terms and conditions.

For the customers in Korea

SONY LIMITED WARRANTY -

Please visit <http://bpeng.sony.co.kr/handler/BPAS-Start> for important information and complete terms and conditions of Sony's limited warranty applicable to this product.

Precaution

On simultaneous use with an electrosurgical knife, etc.

If this unit is used together with an electrosurgical knife, etc., the picture may be disturbed, warped or otherwise abnormal as a result of strong radio waves or voltages from the device. This is not a malfunction.

When you use this unit simultaneously with a device from which strong radio waves or voltages are emitted, confirm the effect of this before using such devices, and install this unit in a way that minimizes the effect of radio wave interference.

On moisture condensation

If the unit is suddenly taken from a cold to a warm location, or if ambient temperature suddenly rises, moisture may form on the outer surface of the unit and/or inside of the unit.

This is known as condensation. If condensation occurs, turn off the unit and wait until the condensation clears before operating the unit. Operating the unit while condensation is present may damage the unit.

On parts life expectancy

The life expectancy of the electrolytic capacitor is about 5 years under normal operating temperatures and normal usage (8 hours per day; 25 days per month). If usage exceeds the above normal usage frequency, the life expectancy may be reduced correspondingly.

Overview

This unit is a video signal input adaptor for Sony monitors.

When installed in an input option slot on the main unit, it provides video input and output connectors for the main unit and a decoder for HD/D1 serial digital component signals.

Notes

- There are some monitors that are not compatible with this unit.
To check if the monitor is compatible with this unit, refer to the instructions for use for the monitor.
- The model name displayed on the monitor's STATUS menu is "BKM-243HS."

Function

Decoder for serial digital component signals

The BKM-243HSM is equipped with a built-in decoder for serial digital component signals.

Serial digital input and output signal connector

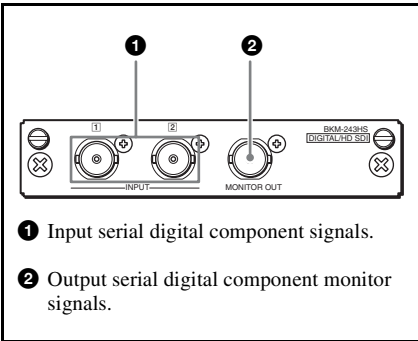
The BKM-243HSM is equipped with two input and one monitor output connectors for serial digital signals.

Using the Input and Output Connectors

For information about installing this unit in an input option slot, refer to the instructions for use for the monitor.

Configuration of input/output connectors and signals that may be input

The configuration of the input and output connectors, and the signals that may be input are shown below.

Input of serial digital component signals

You can input serial digital signals to connectors 1 and output signals displayed on the screen of the main unit or the monitor from connector 2.

You need not attach the 75Ω termination to connector 2.

Notes

- The MONITOR OUT connector outputs signal only when the power of the main unit is ON.
The MONITOR OUT signal is not output when the monitor is in standby mode.
- The signal from the MONITOR OUT connector does not comply with ON-LINE output signal specifications.

Specifications

General

Voltage	+3.3 V, +5 V (supplied from the main unit)
Power consumption	Approx. 2 W
Operating conditions	
Temperature	0 °C to 35 °C (32 °F to 95 °F)
Optimum temperature	20 °C to 30 °C (68 °F to 86 °F)
Humidity	0% to 90% (no condensation allowed)
Pressure	700 hPa to 1060 hPa
Storage and transport conditions	
Temperature	-20 °C to +60 °C (-4 °F to +140 °F)
Humidity	0% to 90% (no condensation allowed)
Pressure	700 hPa to 1060 hPa
Maximum external dimensions (w/h/d)	100 × 20 × 162 mm (4 × ¹³ /16 × 6 ¹ /2 inches)
Mass	Approx. 250 g (8.8 oz)

Input/Output connectors

Digital input

BNC × 2, with monitor output connector

For information about available signal formats, see the instructions for use for the main unit to which the adaptor is installed.

Signal characteristics

Digital component signals

Sampling frequency

D1-SDI: Y/R-Y/B-Y: 13.5 MHz
HD-SDI: Y/Pb/Pr: 74.25 MHz

Quantization

10 bits/sample

MONITOR OUT

Output signal amplitude:

800 mVp-p ±10%

Output impedance:

75Ω unbalanced

Transmission distance

D1-SDI: 200 m (approx. 656 ft) max.
(When using 5C-2V coaxial
cables (Fujikura, Inc.) or
equivalent.)

HD-SDI: 100 m (approx. 328 ft) max.
(When using 5C-FB coaxial
cables (Fujikura, Inc.) or
equivalent.)

Note

Not available for analog signal.

Supplied accessories

Instructions for Use (1)

Information for Customers in
Europe (1)

Design and specifications are subject to change
without notice.

Notes

- Always verify that the unit is operating properly before use. SONY WILL NOT BE LIABLE FOR DAMAGES OF ANY KIND INCLUDING, BUT NOT LIMITED TO, COMPENSATION OR REIMBURSEMENT ON ACCOUNT OF THE LOSS OF PRESENT OR PROSPECTIVE PROFITS DUE TO FAILURE OF THIS UNIT, EITHER DURING THE WARRANTY PERIOD OR AFTER EXPIRATION OF THE WARRANTY, OR FOR ANY OTHER REASON WHATSOEVER.
- SONY WILL NOT BE LIABLE FOR CLAIMS OF ANY KIND MADE BY USERS OF THIS UNIT OR MADE BY THIRD PARTIES.
- SONY WILL NOT BE LIABLE FOR THE TERMINATION OR DISCONTINUATION OF ANY SERVICES RELATED TO THIS UNIT THAT MAY RESULT DUE TO CIRCUMSTANCES OF ANY KIND.

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© 2014 Sony Corporation
D1-SDI: 200 m maks. (Kada se koriste koaksijalni kablovi 5C-2V (Fujikura, Inc.) ili ekvivalentni.)
HD-SDI: 100 m maks. (Kada se koriste koaksijalni kablovi 5C-FB (Fujikura, Inc.) ili ekvivalentni.)