# *Digital Capture Unit*

---

Using a Medical Grade Monitor \_\_\_\_\_ **GB**

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Utilización de un Sony Medical Grade Monitor \_\_\_\_\_ **ES**

*UPA-P100MD*

# When Using a Sony Medical Grade Monitor

The UPA-P100MD supports the connection of a Sony Medical Grade Monitor (hereafter “monitor”), and the Back to Live function. Connecting the UPA-P100MD to a monitor with a custom cable, the Back to Live function can display still images for a set length of time when they are captured.

## Note

To use the Back to Live function, you must use a Sony Medical Grade Monitor, as it is equipped with a remote port. If the monitor you are using is not equipped with a remote port, it is not possible to use the Back to Live function.

## Support Monitors

### U.S., Canada


- PVM-14M2MDU
- PVM-14M2MDU

### Europe


- PVM-14M2MDE
- PVM-20M2MDE

## Custom Cable Pin Assignments

To connect a monitor, you need to prepare a custom cable with the following connectors and pin assignments.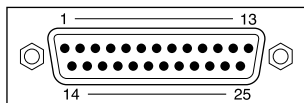

Male D-sub, 25-pin connector

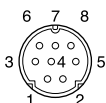Male mini DIN, 8-pin connector

D-sub, 25-pin pin number	Mini DIN 8-pin pin number [wire color]	Signal
1	5 [Green]	STROBE (TALLY LAMP)
14	1 [Brown]	AUTO FEED# (MONITOR REMOTE ON/OFF)
17	4 [Yellow]	SLIN# (MONITOR LINE SELECT)
18 to 25	3 [Orange]	GND (MONITOR)


## Notes

- Use a 4-core shielded cable for the custom cable.
- Make sure that the cable shield is connected to the connector shells at both ends.


## Connecting a Sony Medical Grade Monitor

Connecting a monitor to the UPA-P100MD allows you to monitor the images provided by the video device and use the Back to Live function. Connect the monitor as illustrated below.

- When the video device outputs S-video (Y/C) signals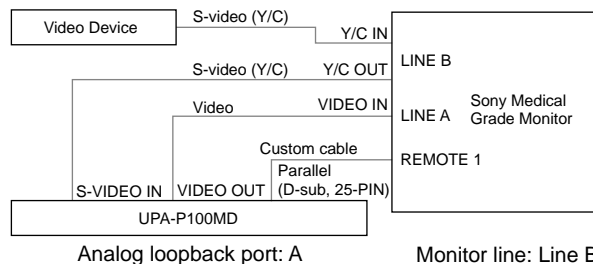

- When the video device outputs video signals

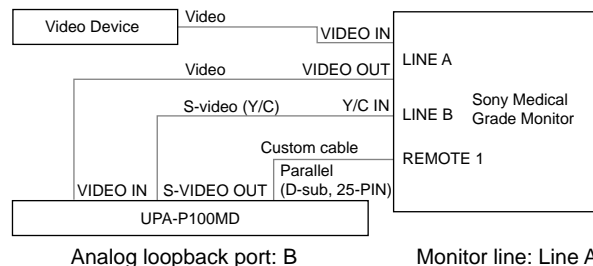## Notes

- When a monitor is connected to the UPA-P100MD with the custom cable, its tally lamp turns on when capturing video.
- When the RGB output is connected to a PC monitor, video output from the S-video and video output ports is not possible. Disconnect the PC monitor and restart the UPA-P100MD.

---

## Using Back to Live (With S-video (Y/C) Signals)

The Back to Live function can display still images for a set length of time when they are captured. To use Back to Live, it is necessary to connect the UPA-P100MD to a monitor with the custom cable mentioned above.

### Notes

- The following procedure assumes that the video device outputs S-video (Y/C) signals. Procedural differences between S-video (Y/C) and video signals are mentioned, as necessary.
- This function is not active when capturing video.

- 1** Connect a monitor to the UPA-P100MD as described above.  
Make sure that the monitor line is Line B.

### Note

When the video device outputs video signals, select Line A as the monitor line.

- 2** Access the configuration screen as described in the UPA-P100MD User's Guide.

- 3** Select [A] for [Analog loopback port].

### Note

When the video device outputs video signals, select [B] for [Analog loopback port].

- 4** Specify a value between 0 and 10 seconds for [Back to live delay (0-10)].

### Note

To deactivate the Back to Live function, specify 0.

- 5** Click [Save Config].  
The UPA-P100MD restarts.
- 6** When you are ready to capture still images, press [1] on the RM-P110.  
[Grabbing Image] appears on the RM-P110 display, the captured still image is displayed on the monitor for the set length of time, then it switches back to the "live" display.

- 7** Capture other still images as necessary.

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