Trinitron® *Color Video Monitor*

Instructions for Use _____ **GB**

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Manual de instrucciones _____ **ES**

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Trinitron

PVM-14L1MDE

Owner's Record

The model and serial numbers are located at the rear. Record these numbers in the spaces provided below. Refer to these numbers whenever you call upon your Sony dealer regarding this product.

Model No. _____

Serial No. _____

WARNING

To prevent fire or shock hazard, do not expose the unit to rain or moisture.

Dangerously high voltage are present inside the unit.

Do not open the cabinet. Refer servicing to qualified personnel only.

THIS APPARATUS MUST BE EARTHED.

In the event of a malfunction or when maintenance is necessary, consult an authorized Sony dealer.

For the customers in the U.S.A.

This equipment has been tested and found to comply with the limits for a Class A digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference when the equipment is operated in a commercial environment.

This equipment generates, uses, and can radiate radio frequency energy and, if not installed and used in accordance with the instruction manual, may cause harmful interference to radio communications.

Operation of this equipment in a residential area is likely to cause harmful interference in which case the user will be required to correct the interference at his own expense.

You are cautioned that any changes or modifications not expressly approved in this manual could void your authority to operate this equipment.

For the customers in Canada

This unit has been certified according to Standard CSA C22.2 NO.60601.1.

For the customers in Europe

This equipment has been found to comply with limits for a Class B device pursuant to EN60601-1-2. However, if this equipment does cause harmful interference to other devices, which can be determined by turning this equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Increase the separation between the equipment and other devices.
- Connect the equipment into an outlet on a circuit different from that to which other devices are connected.
- Consult the dealer or an experienced radio/TV technician for help.

ATTENTION – When the product is installed in a rack:

a) Elevated operating ambient temperature

If installed in a closed or multi-unit rack assembly, the operating ambient temperature of the rack environment may be greater than room ambient. Therefore, consideration should be given to installing the equipment in an environment compatible with the manufacture's maximum rated ambient temperature (T_{mra}: 0°C to 40°C (32°F to 104°F)).

b) Reduced air flow

Installation of the equipment in a rack should be such that the amount of air flow required for safe operation of the equipment is not compromised.

c) Mechanical loading

Mounting of the equipment in the rack should be such that a hazardous condition is not achieved due to uneven mechanical loading.

d) Circuit overloading

Consideration should be given to the connection of the equipment to the supply circuit and the effect that overloading of circuits might have on overcurrent protection and supply wiring.

Appropriate consideration of equipment nameplate ratings should be used when addressing this concern.

e) Reliable earthing

Reliable earthing of rack-mounted equipment should be maintained. Particular attention should be given to supply connections other than direct connections to the branch circuit (e.g., use of power strips).

f) Gap keeping


Upper and lower gap of rack-mounted equipment should be kept 44 mm (1 3/4 inches).

WARNING

Important safeguards/notices for use in the medical environments

- 1 All the equipments connected to this unit shall be certified according to Standard IEC601-1, IEC950, IEC65 or other IEC/ISO Standards applicable to the equipments.
- 2 When this unit is used together with other equipment in the patient area*, the equipment shall be either powered by an isolation transformer or connected via an additional protective earth terminal to system ground unless it is certified according to Standard IEC601-1 and IEC601-1-1.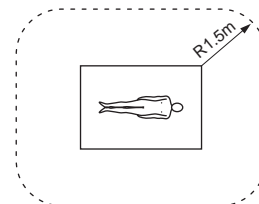

* Patient Area

- 3 The leakage current could increase when connected to other equipment.
- 4 The operator should take precautions to avoid touching the rear panel input and output circuitry and the patient at the same time.
- 5 Model PVM-14L1MDE is a video monitor intended for use in a medical environment to display video pictures from cameras or other video system. These models are nonpatient care equipment with respect to the requirement of leakage current.

Table of Contents

Symbols on the unit

Symbol	Location	This symbol indicates
	Front panel	Main power switch. Press to turn the monitor on or off.
	Rear panel	The equipotential terminal which brings the various parts of a system to the same potential.
	Rear panel	Alternating current
	Rear panel	Attention, consult ACCOMPANYING DOCUMENTS

Caution

When you dispose of the unit or accessories, you must obey the law in the relative area or country and the regulation in the relative hospital.

Precaution	4 (GB)
Features	4 (GB)
Connections	5 (GB)
How to Connect the AC Power Cord	5 (GB)
How to Connect a Cable to a BNC Connector	5 (GB)
Location and Function of Parts and Controls	6 (GB)
Control Panel	6 (GB)
Rear Panel	7 (GB)
Selecting the Menu Language	8 (GB)
Using the Menu	9 (GB)
Adjustment Using the Menus	10 (GB)
Items	10 (GB)
Adjusting and Changing the Settings	10 (GB)
Attaching the Side Covers	11 (GB)
Specifications	12 (GB)
Dimensions	i

Warning on power connection

Use a proper power cord for your local power supply.

	United State	Canada	Continental Europe
Plug type	HOSPITAL GRADE	HOSPITAL GRADE	LP-34A
Female end	E41395	(Classe de qualité hôpital)	LS-60
Cord type	E41395-A	LL33182 LL76662	H05VV-F
Minimum cord set rating	10 A/125 V	10 A/125 V	10 A/250 V
Cord length	Max. 4.5 m (177 1/4 in.)	Max. 4.5 m (177 1/4 in.)	–
Safety approval	UL	CSA	VDE

Precaution

On safety

- Operate the unit only with a power source as specified in “Specifications” section.
- The nameplate indicating operating voltage, power consumption, etc., is located at the rear.
- Should any solid object or liquid fall into the cabinet, unplug the unit and have it checked by qualified personnel before operating it any further.
- Do not drop or place heavy objects on the power cord. If the power cord is damaged, turn off the power immediately. It is dangerous to use the unit with a damaged power cord.
- Unplug the unit from the wall outlet if it is not to be used for several days or more.
- Disconnect the power cord from the AC outlet by grasping the plug, not by pulling the cord.
- The socket-outlet shall be installed near the equipment and shall be easily accessible.

On installation

- Allow adequate air circulation to prevent internal heat build-up.
Do not place the unit on surfaces (rugs, blankets, etc.) or near materials (curtains, draperies) that may block the ventilation holes.
- Do not install the unit in a location near heat sources such as radiators or air ducts, or in a place subject to direct sunlight, excessive dust, mechanical vibration or shock.

On cleaning

To keep the unit looking brand-new, periodically clean it with a mild detergent solution. Never use strong solvents such as thinner or benzine, or abrasive cleansers since they will damage the cabinet. As a safety precaution, unplug the unit before cleaning it.

On repacking

Do not throw away the carton and packing materials. They make an ideal container which to transport the unit.

On Mounting on a Rack

Leave 1U space empty above and below the monitor to ensure adequate air circulation or install a fan to maintain the monitor’s performance.

If you have any questions about this unit, contact your authorized Sony dealer.

Features

Picture

Screen resolution

The use of a Trinitron¹⁾ picture tube, with over 600 lines of horizontal resolution in the center of the picture, provides a clear, vivid picture.

Comb filter

The use of a comb filter allows more accurate Y/C color separation with NTSC video signals. This contributes to less of a decrease in resolution, and prevents the cross color and cross luminance phenomena.

Beam current feedback circuit

The built-in beam current feedback circuit assures stable white balance.

Two color systems available

The monitor can display NTSC and PAL signals. The appropriate color system is selected automatically.

Input

Y/C input connector (S-input connector)

A video signal, split into a luminance component (Y) and a chrominance component (C), can be input through this connector, eliminating the interference between the two components, ensuring picture quality.

Automatic termination (only for connectors with a mark)

The input connector is terminated at 75 ohms internally when nothing has been connected to the output connector. If a cable is connected to the output connector, the internal terminal is automatically released and the signals input to the input connector are output to the output connector (loop-through).

.....
1) Trinitron

“Trinitron” is a registered trademark of Sony Corporation.

Functions

16:9 mode

You can precisely monitor a signal whose aspect ratio is 16:9, in addition to a 4:3 signal.

Note

With a 16:9 mode display, red, blue and green scanning lines can be seen at the upper edge of the screen, but this is due to internal operations of the unit itself.

On-screen menus

You can set monitor operation settings by using the on-screen menus.

Side covers

The monitor can be covered with side covers. The side covers that protect the ventilation holes from splashes (of medicines, etc.) as much as possible.

Options

EIA 19-inch rack mount bracket available


The monitor can be mounted on an EIA-standard 19-inch rack, using the following mounting brackets or slide rails.

MB-502C (Outside Europe, use the MB-502B)

For details on mounting the monitor on the rack, refer to the user's manual of the mounting bracket or slide rail.


How to Connect the AC Power Cord

Plug the AC power cord into the AC IN socket.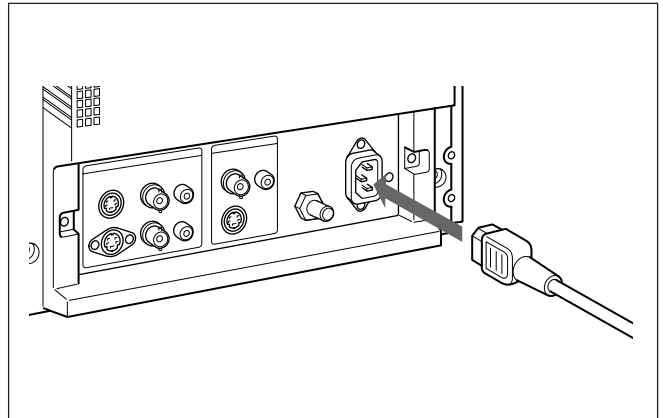


How to Connect a Cable to a BNC Connector

Connect the coaxial cable with the BNC plugs to the BNC connectors on the rear panel as illustrated below.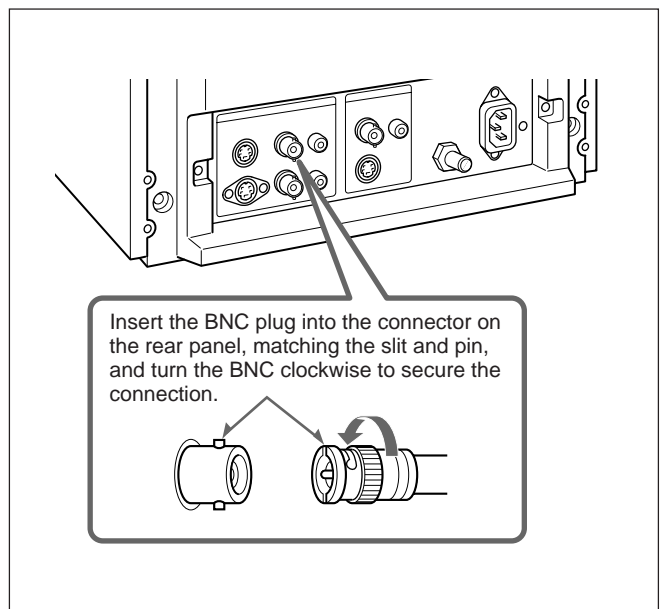

Location and Function of Parts and Controls

Control Panel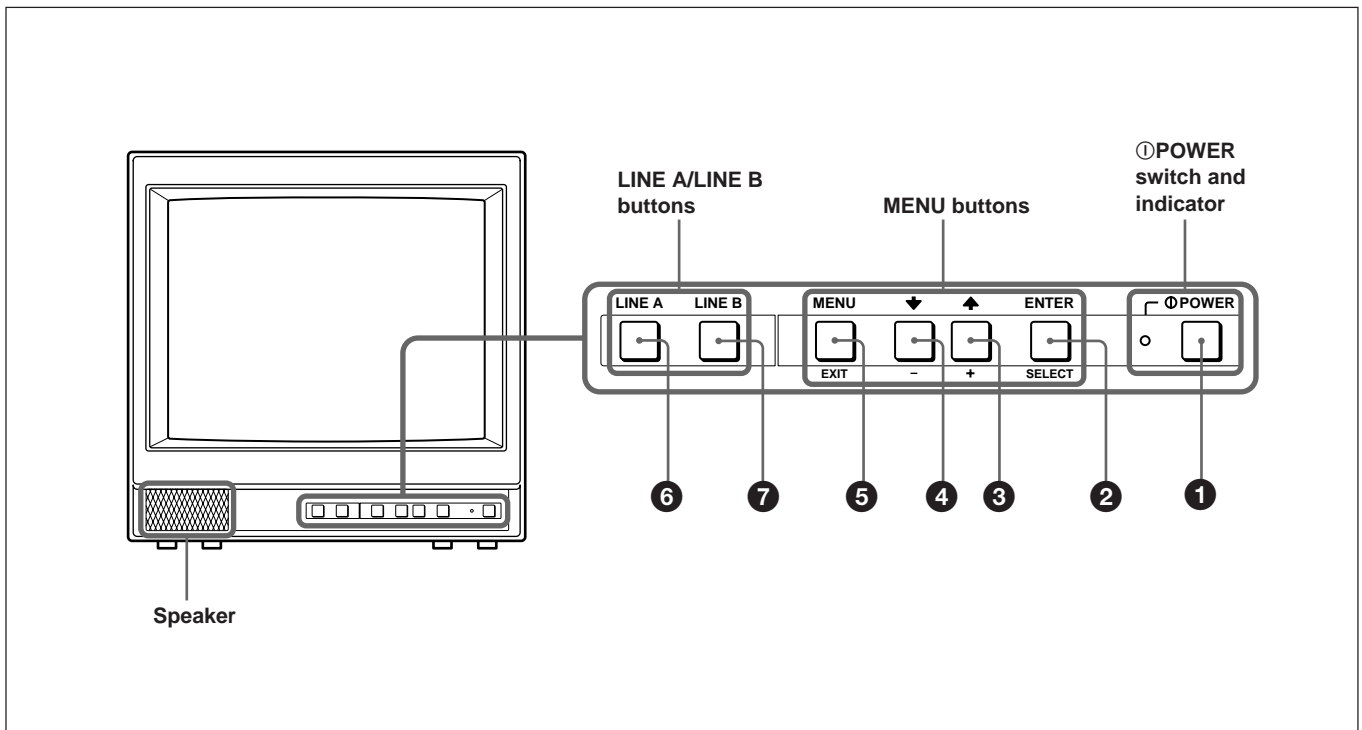

① POWER switch and indicator

① POWER switch

Press this switch to turn on the monitor. The lamp will light up. Press this switch again to turn off the monitor.

MENU buttons

② ENTER/SELECT button

Press this button to confirm an item selected on the menu.

③ ↑/+ (move the cursor up/adjust the value) button

④ ↓/- (move the cursor down/adjust the value) button

Press these buttons to move the cursor or adjust an item selected on the menu.

⑤ MENU/EXIT button

Press this button to show or hide on-screen menus.

LINE A/LINE B buttons


⑥ LINE A button

Press this button to monitor the signal input through the LINE A connectors.

⑦ LINE B button

Press this button to monitor the signal input through the LINE B connectors.

Rear Panel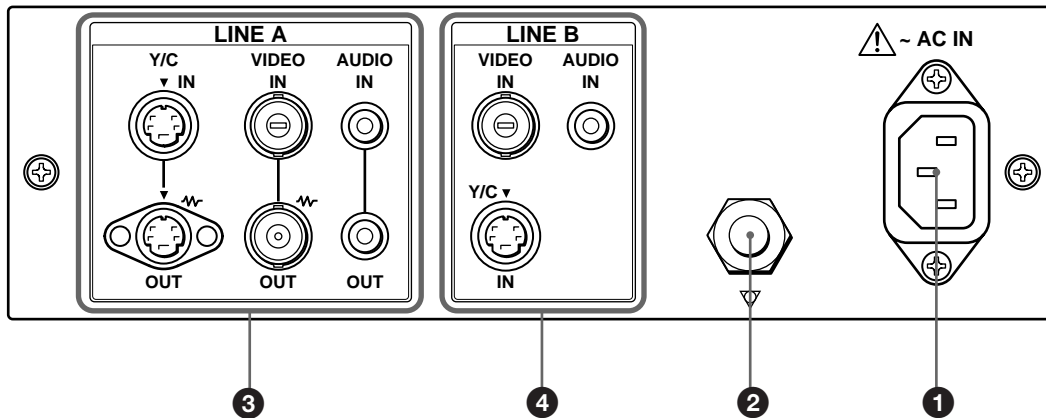

(The \sim mark indicates automatic termination.)

① ~AC IN socket

Connect the supplied AC power cord to this socket and then to a wall outlet.

② Ground (⏏) terminal

Connect a GND cable.

③ LINE A connectors

Line input connectors for Y/C separate, composite video and audio signals and their loop-through output connectors.

Press the LINE A button on the control panel to monitor the input signal through these connectors. If you input signals to both Y/C IN and VIDEO IN, the signal input to the Y/C IN is selected.

Y/C IN/OUT (4-pin mini-DIN)

These are the input/output connectors for a Y/C separate signal. Connect them to the Y/C separate input/output connectors on equipment such as a VCR, video camera, or another monitor.

VIDEO IN/OUT (BNC)

These are the input/output connectors for a composite video signal. Connect them to the composite video input/output connectors on equipment such as a VCR, video camera, or another monitor.

AUDIO IN/OUT (phono jack)

These are the input/output jacks for an audio signal. Connect them to the audio input/output jacks on equipment such as a VCR.

(Continued)

④ LINE B connectors

Line input connectors for composite video and audio signals. Press the LINE B button on the control panel to monitor the signal input through these connectors. If you input signals to both Y/C IN and VIDEO IN, the signal input to the Y/C IN is selected.

Y/C IN (4-pin mini-DIN)

This is the input connector for a Y/C IN separate signal. Connect it to the Y/C separate output connector on equipment such as a VCR, video camera, or another monitor.

VIDEO IN (BNC)

This is the input connector for a composite video signal. Connect it to the composite video output connector on equipment such as a VCR, video camera, or another monitor.


AUDIO IN (phono jack)

This is the input jack for an audio signal. Connect it to the audio output jack on equipment such as a VCR.

Selecting the Menu Language


You can select one of six languages (English, German, French, Italian, Spanish, Japanese) for displaying the menus and other on-screen messages.

The factory preset language is ENGLISH (English). The current settings are displayed in place of the ■ marks on the illustrations of the menu screen.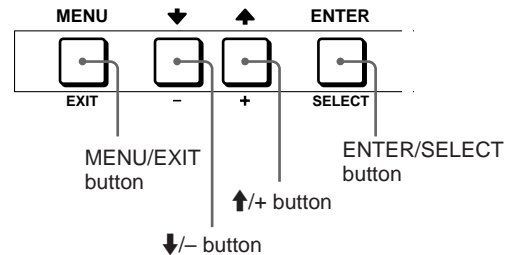


- 1 Press the MENU/EXIT button to display the menu screen, and press the ↑/+ or ↓/- button to select  (USER CONFIG), then press the ENTER/SELECT button.

The USER CONFIG menu appears.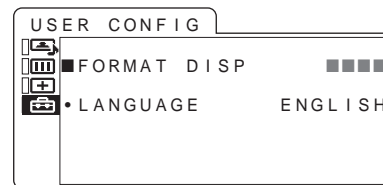

- 2 Press the ↑/+ or ↓/- button to select "LANGUAGE," then press the ENTER/SELECT button.

The selected item is displayed in yellow.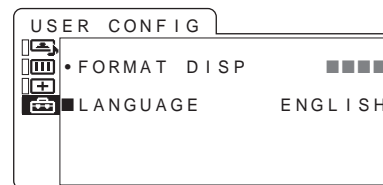

- 3 Press the ↑/+ or ↓/- button to select the desired language, the on-screen language changes to the language you have selected, then press the ENTER/SELECT button to finalize the setting.

To clear the menu

Press the MENU/EXIT button.

The menu disappears automatically if a button is not pressed within one minute.

Using the Menu


The monitor is equipped with an on-screen menu for making various adjustments and settings such as picture control, input setting, set setting change, etc. Follow the instructions below to make adjustments or to change settings.

For details on the menu items, see “Adjustment Using the Menu” on page 10 (GB).


You can also change the menu language displayed in the on-screen menu.

To change the menu language, see “Selecting the Menu Language” on page 8 (GB).


The current settings are displayed in place of the ■ marks on the illustrations of the menu screen.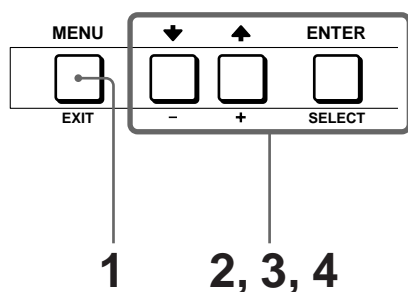

- 1 Press the MENU/EXIT button.
The menu appears.
The menu presently selected is indicated by a yellow cursor.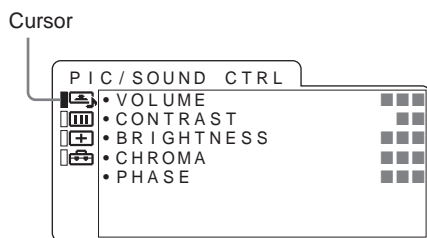

- 2 Press the ↑/+ or ↓/- button to select a menu, then press the ENTER/SELECT button.
The menu icon presently selected is shown in yellow and the available setting items are displayed.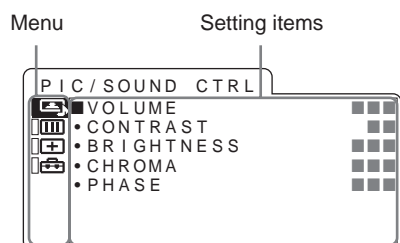

- 3 Use the ↑/+ or ↓/- button to select the desired item, then press the ENTER/SELECT button.
The item to be changed is displayed in yellow.

Note

Based on the item selected, a further level of menus below that of the item selected may appear.

- 4 Make the setting or adjustment in an item.
When changing the adjustment level:
To increase the number, press the ↑/+ button.
To decrease the number, press the ↓/- button.
Press the ENTER/SELECT button to confirm the number, then restore the original screen.
When changing the setting:
Press the ↑/+ or ↓/- button to change the setting.
Press the ENTER/SELECT button to confirm the setting.
The value adjusted in AV control is confirmed even if you does not press the ENTER/SELECT button.

Note

An item displayed in blue cannot be accessed. You can access the item if it is displayed in white.

To clear the menu

Press the MENU/EXIT button.
The menu disappears automatically if a button is not pressed within one minute.


About retaining the settings

The settings are automatically stored in the monitor memory.

Adjustment Using the Menus

Items

The screen menu of this monitor consists of the following items.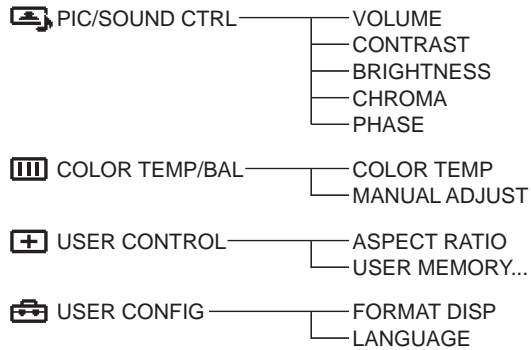

Adjusting and Changing the Settings

PIC/SOUND CTRL menu

The PIC/SOUND CTRL menu is used for adjusting the volume and picture.

Items that cannot be adjusted depending on the input signal are displayed in blue.

Submenu	Setting
VOLUME	Adjusts the audio level.
CONTRAST	Adjusts the picture contrast.
BRIGHT	Adjusts the picture brightness.
CHROMA	Adjusts the color intensity. The higher the setting, the greater the intensity. The lower the setting, the lower the intensity.
PHASE	Adjusts color tones. The higher the setting, the more greenish the picture becomes. The lower the setting, the more purplish the picture becomes.

COLOR TEMP/BAL menu

The COLOR TEMP/BAL menu is used for adjusting the picture white balance.

You need to use a measurement instrument to adjust the white balance.

Submenu	Setting
COLOR TEMP	Select the color temperature from among D65, D93 and USER setting.
MANUAL ADJUST	If you set COLOR TEMP to USER, the item displayed is changed from blue to white, which means you can adjust the color temperature. <ul style="list-style-type: none"> • ADJUST GAIN...: Adjusts the color balance RGB (GAIN). • ADJUST BIAS...: Adjusts the color balance RG (BIAS). • COPY FROM: If you select D65 or D93 with the \uparrow/+ or \downarrow/- button, the white balance data of the selected color temperature will be copied to USER.

USER CONTROL menu

The USER CONTROL menu is used for selecting the aspect ratio, storing or recalling adjustments and settings.

Submenu	Setting
ASPECT RATIO	Select the aspect ratio of the screen between 4:3 or 16:9.
USER MEMORY	To store or to recall all adjustments and settings. <ul style="list-style-type: none"> • STORE: To store all adjustments currently set on AV control menu into the internal memory. • RECALL: To recall all adjustments and settings currently stored in the internal memory.

USER CONFIG menu

The USER CONFIG menu is used to select a language for the menus and the on-screen messages or to select a format display.

Submenu	Setting
FORMAT DISP	Determines whether the format of a input signal is displayed on the screen or not. <ul style="list-style-type: none"> ON: The format is always displayed. OFF: The format is always hidden. AUTO: The format is displayed for about 10 seconds when the input of the signal begins.

Attaching the Side Covers


Submenu	Setting
LANGUAGE	You can select the desired language for the menus or messages from the following language options. 日本語: Japanese ENGLISH: English DEUTSCH: German FRANÇAIS: French ITALIANO: Italian ESPAÑOL: Spanish

Using the Last Control Function

If you press the **↑/+** or **↓/-** button when the menu is not displayed, one of the following menu items that you adjusted last time is displayed.

- VOLUME
- CONTRAST
- BRIGHT
- CHROMA
- PHASE

Then you can adjust the item immediately.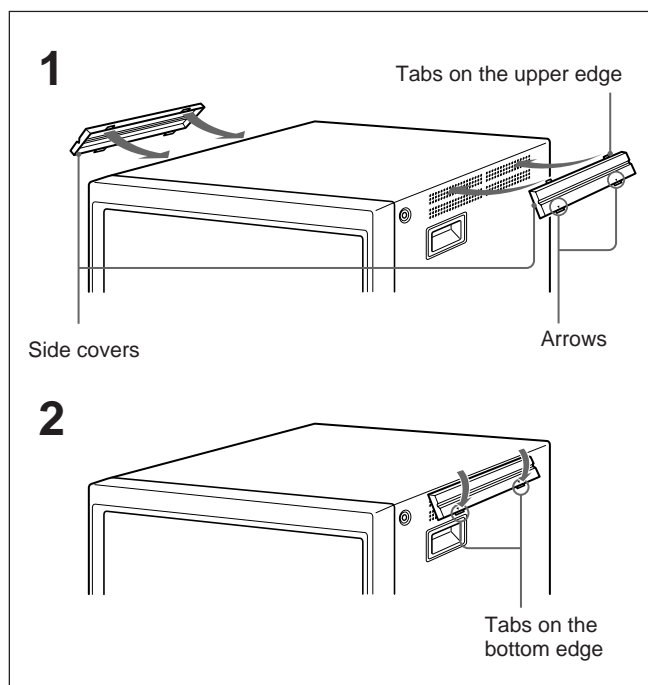

In order to protect the ventilation holes from splashes from medicines, etc., attach the supplied side covers as illustrated.

- 1** Hook the tabs on the upper edge into the ventilation holes, making sure that the arrows on the cover are facing down.

Note

Attach the side covers on all ventilation holes.

- 2** Push up the tabs on the bottom edge and fit the cover into the lowest ventilation holes. Attach covers on both left and right vents.

Specifications

Video signal

Frequency response
LINE 8 MHz +0 dB, -6 dB (Y signal)

Picture performance

Normal scan 7 % over scan of CRT effective screen area
H. linearity Less than 8.0 % (typical)
V. linearity Less than 7.0 % (typical)
Color temperature D65, D93, USER (Adjustable color temperature: 5000 K to 10000 K)
Resolution 600 TV lines (When Y/C signal is inputted)

Input connectors

LINE A/B input connector
Y/C input 4-pin mini-DIN (×2)
See the pin assignment on the page 13 (GB).
VIDEO input BNC connector (×2), 1Vp-p +3 dB, -6 dB, negative synchronization
AUDIO input
Phono jack (×2), -5 dBu^{a)}, 47 kΩ or higher

Output connectors

LINE A output connectors
Y/C output 4-pin mini-DIN (×1) Loop-through, 75 Ω automatic terminal function
VIDEO output
BNC type (×1) Loop-through, 75 Ω automatic terminal function
AUDIO output
Phono jack (×1) Loop-through
Built-in speaker output
0.8 W (monaural)

General

CRT Trinitron P22 luminescent material
Power consumption 68 W, 1.1 to 0.4A
Power requirement 100 to 240 V AC, 50/60Hz
Peak inrush current
(1) Power ON, current probe method: 93 A (240 V)
(2) Hot switching inrush current, measured in accordance with European standard EN55103-1: 42 A (230 V)

Operating conditions

Temperature 0 to +40°C (32°F to 104°F)
Humidity 30 to 85% (no condensation)
Pressure 700 hPa to 1060 hPa

Storage and transport conditions

Temperature -10 to +40°C (14°F to 104°F)
Humidity 0 to 90%
Pressure 700 hPa to 1060 hPa

Dimensions (w/h/d)

Approx. 346 × 340 × 414 mm
(13 ⁵/₈ × 13 ¹/₂ × 16 ³/₈ inches)


Mass Approx. 15 kg (33 lb 2 oz)
Accessory supplied AC power cord (1)
Side covers (2)
Operating Instructions (1)

Design and specifications are subject to change without notice.

a) 0 dBu = 0.775 V r.m.s.

Pin assignment

Y/C IN connector (4-pin mini-DIN)

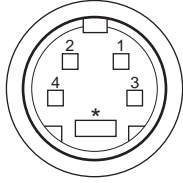Pin number	Signal	Description
1	Y-input	1 Vp-p, sync negative, 75 ohms
2	CHROMA-input Subcarrier-input	286m Vp-p (NTSC), 300m Vp-p (PAL), burst Delay time between Y and C: within 0 ± 100 nsec., 75 ohms
3	GND for Y-input	GND
4	GND for CHROMA- input	GND

Classification of equipment

- Evaluated to EN60601-1, EN60601-1-2, 93/42/EEC
- EQUIPMENT not suitable for use in the presence of a FLAMMABLE ANAESTHETIC MIXTURE WITH AIR or WITH OXYGEN OR NITROUS OXIDE
- Equipment energized from an external electrical power source Class I Equipment
- Degree of protection against electric shock No applied part
- Mode of operation Continuous operation

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